

# **STANDARD OPERATING PROCEDURES**

5. **To monitor respiration rate**, connect gray line jack (V3422) to unit, connect sensor to airway adapter, and place airway adapter between endotracheal tube and wye-piece of anesthesia machine (see **SOP #1102.1**).
6. **To monitor body temperature**, insert black jack (C400-10) into unit and connect temperature probe into other end. Disinfect temperature probe prior to each use.

#### IV. EQUIPMENT USE

1. Turn on unit **STANDBY/ON** by pressing the **• /'** button.
2. **WAVE/TREND** key controls waveform display trend or plethysmogram.
3. To silence the **ALARM**, press the **°** button to silence for two minutes. Holding button down for 3 seconds disables the alarm indefinitely. Press the button again to cancel alarm silence.
4. **MENU/ENTER** key **□/◇** will display the menu list, press again while the menus are displayed to select a menu or accept a value that has been adjusted.
5. **▼/▲ ARROWS** will move among menu items in menu display mode. If trend is displayed, Arrow Keys cycle between trended parameters. If plethysmogram is displayed, Arrow Keys adjust the display's contrast.
6. **To measure oxygen saturation**, place sO<sub>2</sub> sensor clip/probe on patient. If sensor

SOP #1144.2  
SurgiVet V3395–Pulse Oximeter Patient Monitor